



**Hygienekontrolltechnik, Reinhold Pfeiffer**  
Chemnitzer Straße 33, D-09212 Limbach-Oberfrohna, German  
Tel./Fax +49 (0)3722-95370, E-mail: HK-Pfeiffer@web.de

## Cleaning indicator for surface cleaning - CLEANtest K indicators

### Product information

<b>Field of application:</b>	Indicator for self-monitoring of the cleaning effect of even surface at general cleaning and in washer-disinfectors	
<b>Benchmark standard:</b>	DIN ISO/TS 15883-5	
<b>Specifications:</b>	Test soiling:	coloured (red) modified RAMS*
	Substrate/carrier:	self-adhesive transparent film dots
	Test:	pass- fail detection at surface
	Packaging unit:	108 cleaning indicators on two sheets of 54 cleaning indicators each
	Shelf life:	12 months with correct storage
	Storage:	room temperature, protected from light, relative humidity not more than 65%
<b>Instructions for use:</b>	The indicators are affixed to critical surface locations. After completion of the cleaning process, the indicators are evaluated for residual soiling. If no coloured test soiling is visible any more, the cleaning process was successful. The indicators can be reaffixed for providing evidence. The precise procedure for self-monitoring depends on the test objective and is described in the package insert.	

Subject to technical modifications.